Användarmanual ZenSeat® App

Larm-vy

Denna vyn visar pågående larm.

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The screenshot shows the zenseat® interface for 'Vårdboende123'. It is divided into two main sections: 'Patientlarm' and 'Utrustningslarm'. The 'Patientlarm' section lists four alarms for different rooms: Rum 128 (red icon, 3 days 8 hours 21 minutes, with a green 'Medicinerat' tag), Rum 216 (red icon, 22 min), Rum 122 (orange icon, 2 days 6 hours 34 minutes), and Rum 122 (orange icon, 1 day 2 hours 31 minutes). The 'Utrustningslarm' section lists two alarms: Rum 128 (blue icon, 18% battery, Toalett 00878) and Rum 245 (blue icon, Toalet 123467-980+ aB akjss, 2024-05-02 16:02). Each alarm entry has a right-pointing arrow.

Room	Icon	Duration / Status
Rum 128	Red (Stool)	3 dygn 8 timmar 21 minuter (Medicinerat)
Rum 216	Red (Toilet)	22 min
Rum 122	Orange (Stool)	2 dygn 6 timmar 34 minuter
Rum 122	Orange (Stool)	1 dygn 2 timmar 31 minuter
Rum 128	Blue (Battery)	18% (Toalett 00878)
Rum 245	Blue (Toilet)	2024-05-02 16:02 (Toalet 123467-980+ aB akjss)

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

The screenshot displays the 'zenseat' app interface. At the top, the logo and the name 'Valltorp' are visible. Below this is a list of rooms: 'Rum 201', 'Rum 202', 'Rum 210', and 'Rum 118'. Each room entry has a red status indicator on the left and a right arrow on the right. Overlaid on this is a 'Inställningar' (Settings) modal. The modal has a title bar with 'Inställningar' and a 'Logga ut' (Logout) link. Below the title bar is a toggle for 'Notifikationer' (Notifications), which is currently turned on. Underneath are three dropdown menus: 'Vårdboende' (Care home) set to 'Valltorp', 'Språk' (Language) set to 'Svenska' (Swedish), and 'Tidszon' (Time zone) set to 'Europe/Stockholm'. At the bottom of the modal is a 'Spara' (Save) button.

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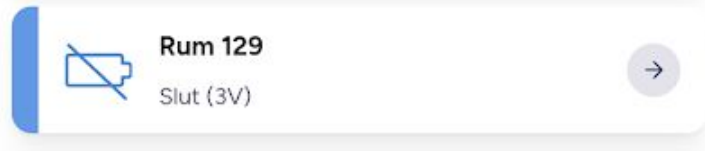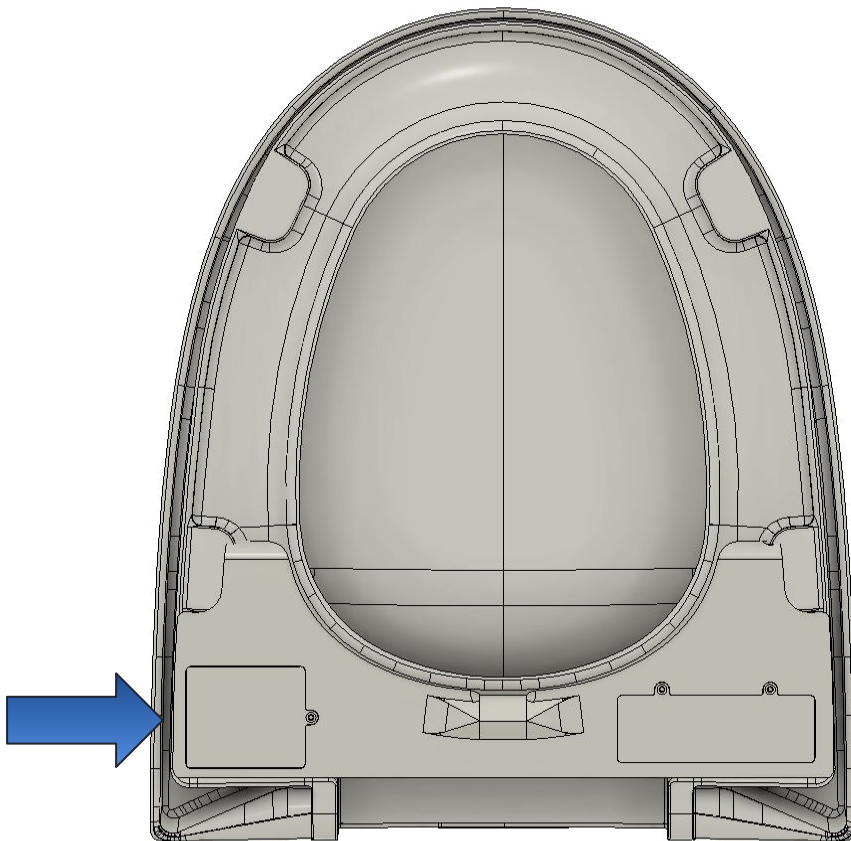Hello!

EC Developer have invited you to join them on ZenSeat®.

Register here

If you do not want to accept, you may safely ignore this email. The invitation expires at 9th June at 13:33.

Regards,
ZenSeat

If you're having trouble clicking the "Register here" button, copy and paste the URL below into your web browser: <https://helleringen.test/join/473a2ced-b1e4-4130-b1c7-9b94e5ed047f>

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